

Instructions

Ribbon® bondable reinforcement

Ribbon-THM, Ribbon Ultra Original Ribbon

Contents

Ribbon starter kits contain:

Ribbon® bondable reinforcement ribbon and scissors.
Ribbon Scissors are only intended for cutting Ribbon fiber.
The scissors are not intended for other purposes.

Ribbon refill kits contain:

Ribbon® bondable reinforcement ribbon.

Statement of use: Ribbon reinforces dental resins.

Handling requirements: Before resin is applied, do not touch Ribbon with hands or latex gloves. Handle Ribbon with tweezers, pliers or cotton gloves.

After resin is applied, handle Ribbon like the resin (instruments or powder free gloves).

Bonding Ribbon to composite: Wet Ribbon with unfilled bonding resin. Imbed Ribbon within uncured filled composite.

Bonding Ribbon to acrylic: Wet Ribbon with a loose mix of liquid and powder. Imbed Ribbon within uncured acrylic.

Polishing Ribbon: Fiber reinforcements do not polish. Cover Ribbon with filled composite or acrylic.

Disposal: No special considerations.

For Professional Use Only

Ribbon® Bondable Reinforcement Ribbon

Made in the USA

US Patent - 9138298 Europe - 2731537

Germany - 6020120439866 - Japan - 特許第 5 9 2 2 7 0 号



Ribbon Inc.
1904 Third Ave Suite 330
Seattle, WA 98101 USA
Tel: 206-340-8870, ribbon@ribbon.com
www.ribbon.com



European Representative:
Sigma Dental Systems
Heideland 22
24976 Handewitt
Germany